

QUICK EASY USE INSTRUCTIONS



1. Rinse your appliance under running water. If needed, use a toothbrush or denture brush to remove any food debris.



2. Place the appliance in a glass with 5 ounces of warm water (90°F - 115°F), ensuring it is fully immersed. Immediately add one tablet to the water. Soak for 15 minutes. The tablet dissolves slowly to allow for gentle and thorough cleaning.



3. Remove the appliance from the solution and rinse thoroughly under water before placing it in your mouth. If a chemical taste remains, soak the appliance in warm water for 10 minutes.



4. **RECOMMENDED TREATMENT**
For optimal results, use daily for the first 21 days to establish a foundation for the product and to support long-lasting freshness. Afterward, continue using the product at least twice weekly to maintain and preserve these benefits.

⚠ Warning

This product contains persulfate, a known allergen. Avoid contact with skin and eyes. Use only as directed. Do not use on natural teeth. Do not mix with other cleaners. Do not place tablets or cleaning solution in the mouth. If ingested, contact the Poison Control Center at 1-800-222-1222. In case of eye contact, rinse thoroughly with water. Discard the solution immediately after use. **Do not reuse the solution.**