



# Dentach<sup>®</sup> Fresh

## Instructions for Using Dentach<sup>®</sup> Fresh Denture Adhesive Cream

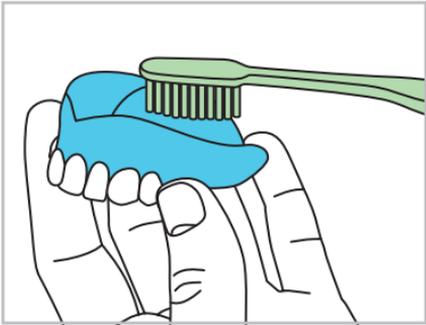


### 1. Prepare Your Dentures

Thoroughly clean and dry your dentures before applying the cream.

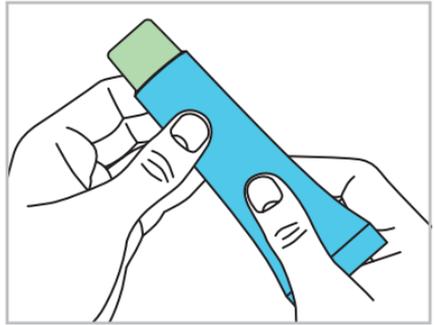


### 2. Mix the Cream

**IMPORTANT!** Knead the tube gently for 20-30 seconds before opening. This blends the natural olive oil, which may separate over time, ensuring smooth, even consistency.

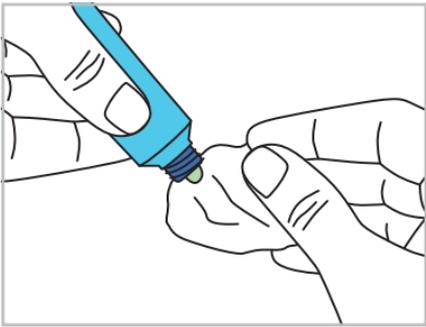


### 3. Handle Oil Residue

If there is oily residue when squeezing out the cream, wipe it off before applying more.

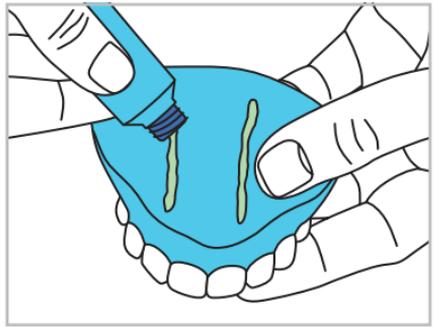


### 4. Apply the Cream

Place the cream on dry dentures as shown in the picture. **Note:** The olive oil gives the cream a unique consistency compared to petrolatum-based adhesives. If you're switching from another product, it may take a week or more to adjust to Dentach<sup>®</sup> Fresh.

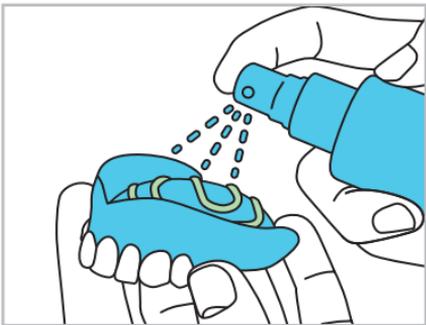


### 5. Moisten Before Inserting

Wet your dentures slightly with water before placing them in your mouth. If you have a dry mouth, take a sip of water to wet your mouth. Remember: Apply the cream to dry dentures and only wet them after applying the cream.

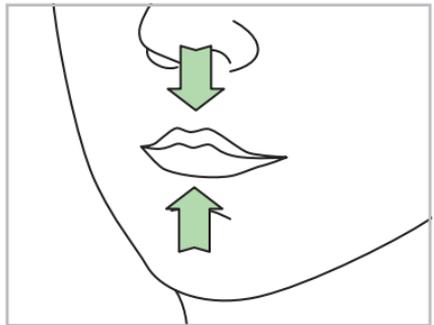

### 6. Insert Dentures

Place the dentures in your mouth, bite down gently, and wait five minutes before eating or drinking.

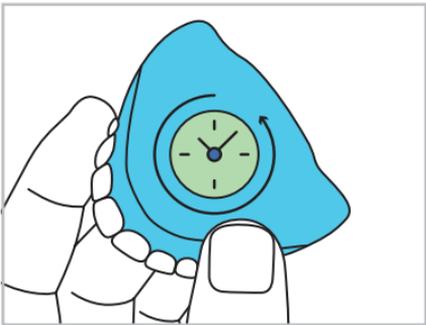

### 7. Removing Your Dentures

After a period of time, the adhesive will naturally loosen. If removal of the adhesive is difficult, the adhesive is still active. Please proceed to Step 8 to remove it. Reduce the amount of cream if you prefer a shorter hold time.



### 8. If Needed, Use Warm Water

If removal is still difficult, rinse your mouth with warm water for 1-2 minutes, letting the water flow between your dentures and gums.

## Important Tips for Use

- Adjust the amount of cream to find your ideal hold time.
- Lower dentures may require more adhesive. Reapply as needed for a secure fit.
- We recommend using Denttach® Clean on the dentures for maximum benefit.
- To remove any remaining cream residues from the denture, brush it with a denture brush or toothbrush using warm water.
- Wipe your gums with a dry paper towel to remove any leftover cream.

## Storage Instructions

- Keep the tube away from heat and direct sunlight. The natural olive-oil formula is sensitive to temperature and may harden if stored in heat.
- If the cream hardens, follow these steps:
  1. Place the tube (cap up) in a glass of hot water for 1 minute, ensuring the tube body is fully submerged.
  2. Remove and gently massage the tube for 30 seconds.
  3. Rinse under cold water while continuing to massage for another 30 seconds to soften the cream.

**CAUTION:** To avoid tube clogging, apply the cream only on dry dentures and keep the nozzle and cap dry at all times. Do not apply on wet dentures. Do not use if the tube nozzle is unsealed, damaged, or appears tampered with. Keep out of reach of children. Do not use if there is a known sensitivity or allergy to any of the ingredients. Do not swallow. If you are taking medication that may affect your gums or oral cavity, consult a dentist before use. If irritation, soreness, or discomfort occurs, discontinue use and consult your dentist. Pain or discomfort may result from ill-fitting or damaged dentures. Consult your dentist regularly to ensure proper fit. Store in a cool, dry place, away from heat, humidity, and direct sunlight, and ensure the cap is tightly closed after each use. Do not use after the expiry date.

## BENEFITS OF DENTTACH® FRESH DENTURE ADHESIVE CREAM

