


Dentach® Fusion


INSTRUCTIONS FOR USE DENTTACH® FUSION DENTURE ADHESIVE CREAM

Product Description


This denture adhesive is designed to provide a strong hold, comfort, and lasting freshness throughout the day. It forms a thin, flexible layer between the denture and oral tissue to improve stability and help reduce food particle infiltration.

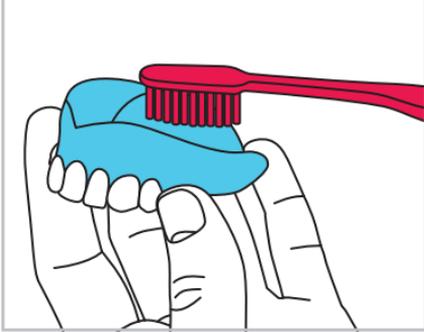

1. Clean and dry your dentures thoroughly before applying.

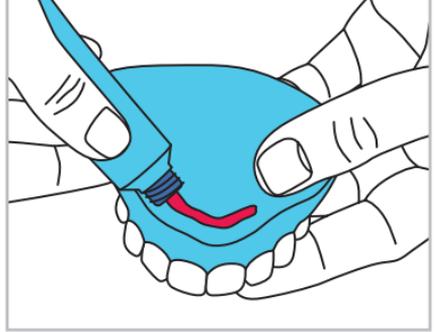


2. Knead the tube with the cap on for 10–30 seconds before each use to ensure a uniform cream. Apply the adhesive to dry dentures, starting with a small amount and adding more if needed.

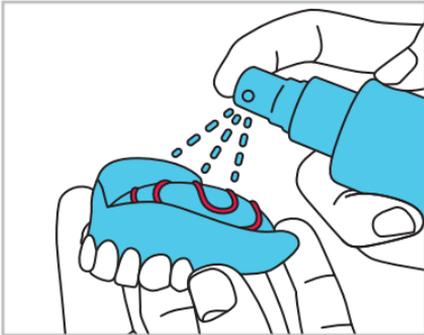

3. Gently moisten your denture with water before placing it in your mouth.




4. Take a sip of water to moisten your gums. This is especially recommended for people with dry mouth, as the adhesive activates with moisture from saliva.

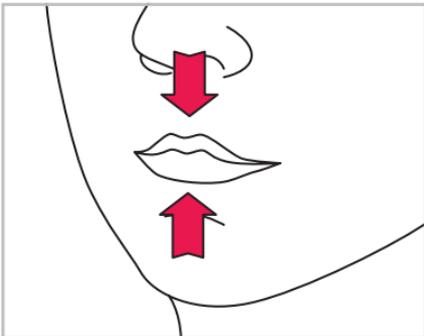


5. Place your dentures in your mouth and hold them in place by biting down and maintaining pressure for a few seconds.



6. Wait 5 minutes before eating or drinking.

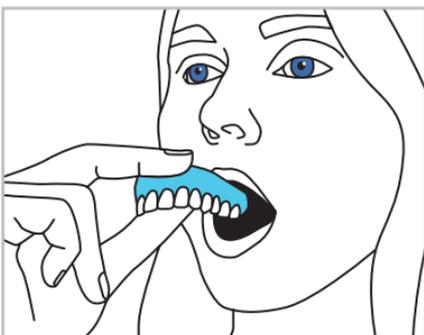

7. To remove, rinse your mouth with warm water and gently rock the denture until it loosens.

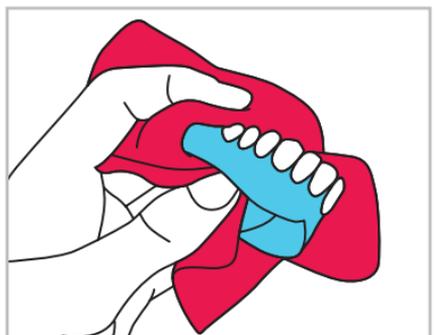

8. Clean your dentures daily with a dry towel to remove residue and then use a denture cleaner such as Dentach® Clean.

INGREDIENTS: Cellulose Gum (44%), Petrolatum, Paraffinum Liquidum, Calcium/Sodium PVM/MA Copolymer (12%), Silica Dimethyl Silylate, Silica, Menthol, Menthyl Lactate, Lecithin.

WARNINGS AND PRECAUTIONS

To avoid tube clogging, apply the cream only on dry dentures and keep the nozzle and cap dry at all times. Do not apply on wet dentures. Use the minimum amount needed for a secure hold and do not use more than once per day. Excess adhesive should be removed from the gums and dentures after use. For dental prosthesis use only. Do not swallow. If swallowed in large quantities, seek medical advice. Keep out of reach of children. Do not use if allergic or hypersensitive to any of the listed ingredients, or if the tube nozzle is unsealed, damaged, or appears tampered with. If you are taking medication that may affect your gums or oral cavity, consult a dentist before use. If irritation, rash, or discomfort occurs, discontinue use immediately and consult a dental professional. Pain or discomfort may result from ill-fitting or damaged dentures. Consult your dentist regularly to ensure proper fit. Store in a cool, dry place, away from heat, humidity, and direct sunlight. Do not freeze. Ensure the cap is tightly closed after each use. Do not use after the expiry date.

Note: If the cream is difficult to squeeze out, place the tube in a glass of warm water (up to 100°F / 38°C) for a short period of time, keeping the cap above the water. If the opening becomes clogged, use a clean pin to clear it. Always keep the tube nozzle dry and free of excess cream to prevent clogging.

BENEFITS



**STRONG LONG
LASTING HOLD**

**ZINC FREE
FORMULA**



**FRESH
TASTE**

**SIMPLE
TO USE**



**NO ARTIFICIAL
COLORING**