



Instructions for Using Denttach® Gold Denture Adhesive Cream

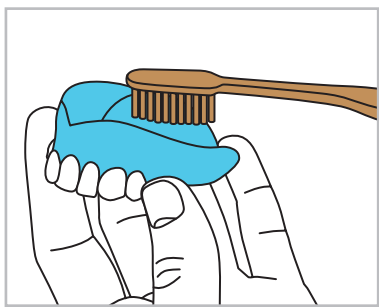


1. Prepare Your Dentures

Thoroughly clean and dry your dentures before applying the cream.

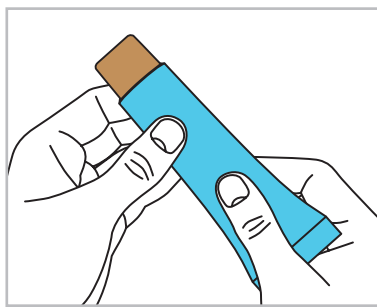


2. Mix the Cream

IMPORTANT! Before opening the tube, knead it gently for 20-30 seconds. This helps mix any natural oil that may have separated, which is normal due to the olive oil base.

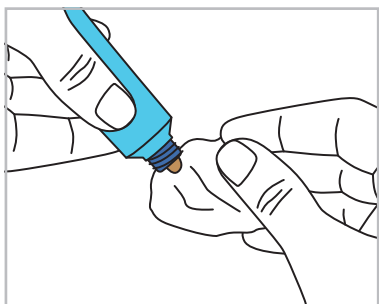


3. Handle Oil Residue

If there is oily residue when squeezing out the cream, wipe it off before applying more.

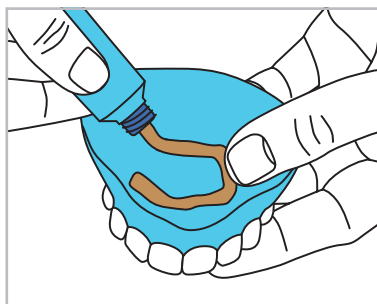


4. Apply the Cream

Place the cream on dry dentures as shown in the picture. **Note:** The olive oil gives the cream a unique consistency compared to petrolatum-based adhesives. If you're used to a petrolatum-based product, it may take a week or more to adjust to Denttach® Gold.

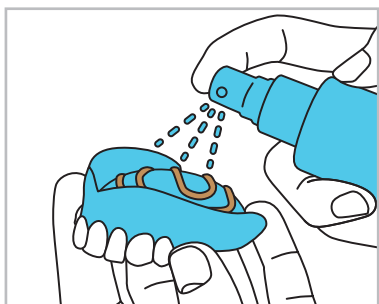


5. Moisten Before Inserting

Wet your dentures slightly with water before placing them in your mouth. If you have a dry mouth, take a sip of water to wet your mouth. Remember: Apply the cream to dry dentures and only wet them after applying the cream.

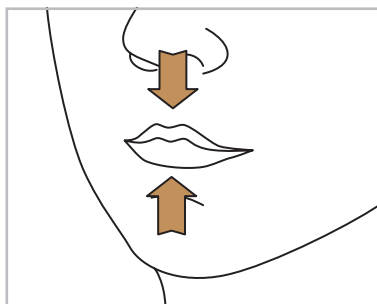


6. Insert Dentures

Place the dentures in your mouth, bite down gently, and wait five minutes before eating or drinking.

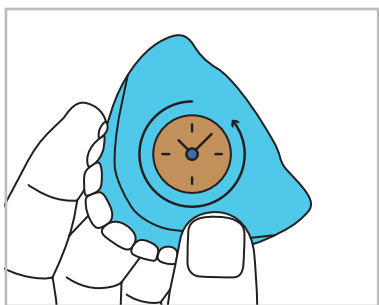

7. Removing Your Dentures

After a long period of time the adhesive will naturally loosen. If you find it difficult to remove them, the adhesion properties are still active. Reduce the amount of cream used if you want a shorter hold time.

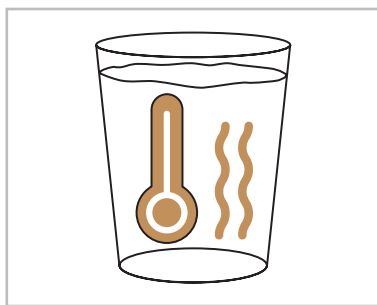

8. If Needed, Use Warm Water

If removal is still difficult, rinse your mouth with warm water for 1-2 minutes. Ensure the water flows between the dentures and your gums.

Important Tips for Use

- Adjust the amount of cream to find your ideal hold time.
- Lower dentures may require more adhesive. Reapply as needed for a secure fit.
- Due to the natural consistency of this product the color may change from tube to tube.
- We recommend using Denttach® Clean on the dentures for maximum benefit.
- To remove any remaining cream residues from the denture, brush it with a denture brush or toothbrush using warm water.
- To remove any remaining cream residues from your gums, use a dry paper towel to wipe the cream residue away.

Storage Instructions


- Keep the tube away from heat and direct sunlight. The natural olive oil formula can be sensitive and may harden if exposed to heat over time.
- If the cream hardens, follow these steps:
 1. Place the tube (cap up) in a glass of hot water for 1 minute, ensuring the body is fully submerged.
 2. Remove and massage the tube for 30 seconds.
 3. Rinse under cold water while massaging for another 30 seconds to soften the cream.

CAUTION: If the tube appears to be tampered with, do not use. Please contact customer service immediately. Use the product according to instructions for use. Keep out of the reach of children. Do not use if there is a known sensitivity or allergy to one of the ingredients. If you are taking medication that may affect your gums or oral cavity, consult a dentist before using the product. In the event of gum irritation or soreness, discontinue use and consult your dentist if necessary. Pain and discomfort of the gums can also be caused by ill-fitting or broken dentures therefore it is important to consult your dentist on a regular basis. Store dry and at room temperature.

BENEFITS OF EXTRA VIRGIN ORGANIC OLIVE OIL




MONOUNSATURATED FATTY ACIDS

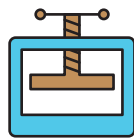


PHYSICAL PRODUCTION PROCESSES



ANTIOXIDANTS: NATURE'S PROTECTORS

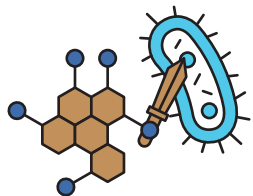

POWER AGAINST MICROBES

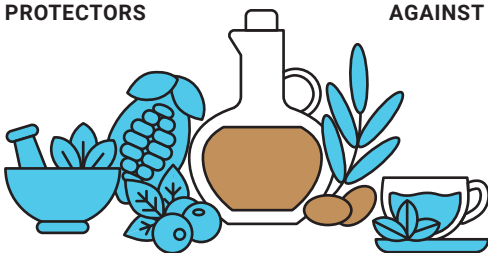

A STEP TOWARD WELLNESS